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- Instructions for Use • Mode d'emploi

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Instructions for Use

ABC Silicone Breast Prostheses

Thank you for choosing ABC Breast Care. ABC products are developed and manufactured with great care and knowledge of the needs of breast-operated women.

ABC Silicone Breast Prostheses & ABC AllSil Breast Prostheses.

This instruction manual DOES NOT apply to:

- ABC Attach Breast Prostheses
- ABC Custom Breast Prosthesis
- ABC Textile Breast Prostheses
- ABC Nipples
- ABC Swim Forms

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(Please note that not all ABC products are available in all countries.)

Scope of delivery:

- ABC Breast Prosthesis
- Travel case for storage and travel
- Instruction manual

1. Intended Use & Indication

- ABC breast prostheses are intended for external use only.
- ABC full and partial breast prostheses (ABC Shapers) are intended for anatomical and weight-based compensation of removed or missing breast tissue. This can be as a result of breast surgeries (such as mastectomy and/or breast-conserving surgery and/or lumpectomy and/or implant removal), breast malformation, breast regression or natural breast asymmetry.
- In addition, they are designed to help restore and maintain anatomical body balance and appearance, prevent shoulder misalignments and allow for natural movement of the breast comparable to the natural breast.
- ABC breast prostheses are intended to be used inside pocketed special bras.

- ABC partial breast prostheses can be worn directly on the skin in a well-fitting bra.
- Shape, size, weight and color of the breast prosthesis are chosen according to the anatomical conditions and needs of the patient.
- The suitable breast prosthesis and special bra are selected and fitted by trained professionals.
- External breast prostheses and special bras are intended for daily use and for wearing over longer periods of time.
- External breast prostheses are intended for adolescent and adult patients from 12 to 99+ years.

2. Risks & Contraindications

- There are no known risks or contraindications to be expected when wearing external ABC AllSil breast prostheses under normal conditions and in accordance with the care instructions. No safety issues or performance impairments are expected during normal use. No restrictions on use are expected.
- Existing allergies should be clarified and ruled out.
- Do not wear your breast prosthesis over open wounds or sensitive skin
- We recommend wearing breast prostheses and partial breast prostheses only after the wound has fully healed and radiotherapy is complete.
- If you experience redness or rashes, please discontinue using the product and contact your doctor or medical device retailer

3. Material

- ABC silicone breast prostheses are made of polyurethane film, silicone gel based on polydimethylsiloxane, colour pigments and microspheres (in lightweight prostheses).
- ABC AllSil breast forms are made of 100% silicone and colour pigments.
- ABC breast forms are carefully checked for quality.
- All materials which have been tested for skin compatibility.

4. Care Instructions

What to avoid:

- Avoid direct contact with perfume, hot surfaces or hot water.
- Avoid sharp objects. Be careful with sharp, pointed objects. Brooches, needles, fingernails, scissors, pet claws, and other sharp objects can cause permanent damage to your breast prostheses.
- Avoid stretching, twisting, friction or excessive use of force when handling your breast prosthesis.
- Your ABC Silicone Breast Prosthesis is resistant to sweat, moisture, water (both salt and chlorine water).
- For regular swimming and other water sports activities, we recommend the additional use of the ABC Active Swim Form.
- Discolouration of the prosthesis, e.g. due to dark clothing or radiation marks, does not affect the function, quality or safety of the breast prosthesis.
- Any changes to the relevant body parts (surgical area, body size, remaining natural breast, etc.) require a new adjustment.
- Regular examination of the fit and positioning of the prosthesis and the special bra by qualified personnel is essential for your comfort and safety.

5. Cleaning Instructions

(The breast prosthesis must not be washed in the washing machine!)

Cleaning instructions for ABC silicone breast forms

- The prosthesis must not be washed in the washing machine!
- Clean your ABC breast prosthesis daily in lukewarm water with mild soap and dry with a soft towel.
- Rinse your prosthesis with lukewarm water after wearing or swimming in salt or chlorinated water.
- Store your breast prosthesis with the cup side down in the ABC travel case.

6. Warranty

- When worn under normal conditions and in compliance with the instructions for use, we guarantee a product service life of two (2) years.
- Should your breast prosthesis become defective, please contact your medical device retailer. In the event of a production or material defect, ABC will replace the breast prosthesis should the defect happen within two (2) years from the date of purchase.

7. Incident Reporting

Only serious incidents relating to the product, which lead to a serious deterioration of the state of health or death, must be reported to the manufacturer or competent authority of the EU Member State. Serious incidents are described in Article 2(65) of Regulation (EU) 2017/745 (MDR).

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Consult instructions for use

9. Additional Information	10. Shipping and Storage Information	11. Disposal
 Medical Device  One Patient Multiple Use  Non-Sterile	 Protect from Sunlight  Store Dry	 Household Waste

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ABC Breast Care GmbH