
Instructions for Use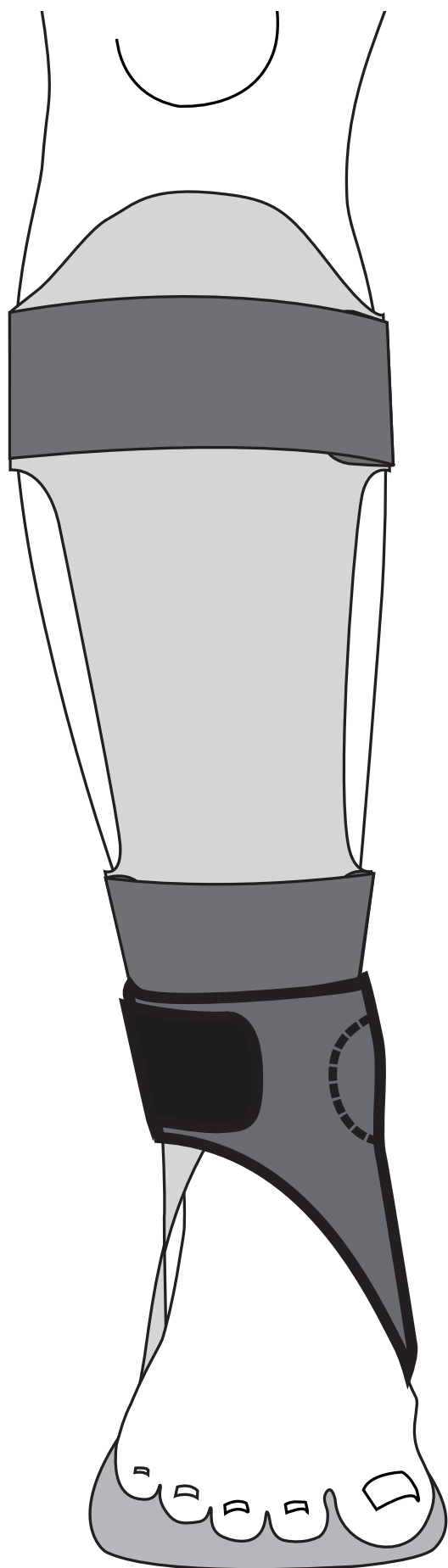

T-Strap

Item No. 28777

EN T-Strap

Intended use

T-Strap is only intended for use in conjunction with an Allard AFO, and together with a custom orthotic appropriate to meet the patient's foot positioning requirements. It is strongly recommended to use T-Strap in conjunction with an orthopedic customized insole/insert on top of the Allard AFO footplate or the AFO may be damaged over time due to excessive stress. Single Patient - multiple use.

Indications

Severe pes valgus that cannot be controlled from ground up with foot orthotics.

Contraindications

Edema, impaired circulation, skin maceration, or other conditions that may be further injured by compression.

Warnings

Ensure the straps are not over tightened. If skin irritation occurs, remove the support, and discontinue use, and report issue to your treating clinician. Check your skin daily.

Material specifications

Main material: Polyurethane, Plastazote & Polyester

Hook/Loop : Nylon

D-ring: POM

This product is not flame retardant

Washing Instruction

Machine wash 40° C, gentle cycle. Dry flat. Do not dry clean or tumble dry.

This product is not flame retardant.

Adjustments

Fitting and modification of this product may only be performed by qualified health care professionals .

Questions

For any questions or issues, please contact your local fitter.

Information

Any serious incident that may occur in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and / or patient is established.







Disposal

Disposal in accordance with local legislation.

Instructions

1. Apply the large patch of self-adhesive hook on the bottom of the footplate where the distal end of the T-Strap is to be attached.
2. Position patient foot in properly aligned position on the Allard AFO orthosis (including orthotic insole and/or any any other orthotic additions normally worn with the orthosis). Position the circle marked on the proximal segment of the T-Strap over the medial malleolus. Wrap the proximal ends of the T-Strap around the ankle toward the lateral strut.
3. Feed the posterior strap through the D-Ring and cinch-close. Mark location of top and bottom of the ankle strap on the lateral strut, using removable marker.
4. Release strap from the D-Ring and apply the small patch of self-adhesive hook to the outside of the lateral strut between your markings.
5. Attach the ankle strap in the proper position by pressing the pile fabric into the hook on the lateral strut. Feed the strap through the D-Ring and cinch-close.
6. Grasp the distal vertical segment, pull down and wrap under footplate. Secure by pressing the pile fabric into the hook on the bottom of the footplate.

NOTE: Observe that the heel height may increase slightly due to the material under the footplate. The material will compress, but not totally.

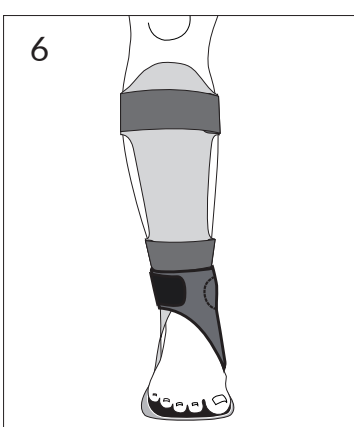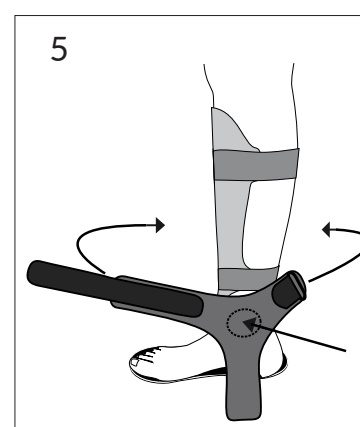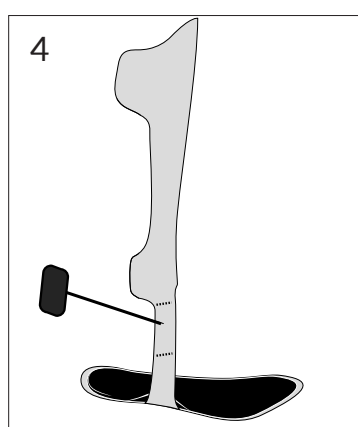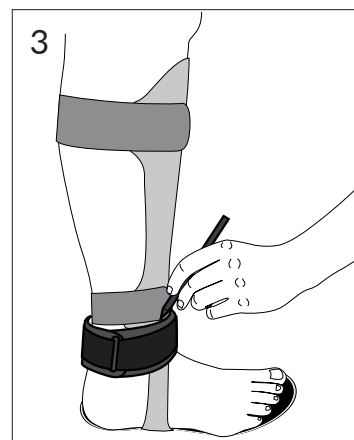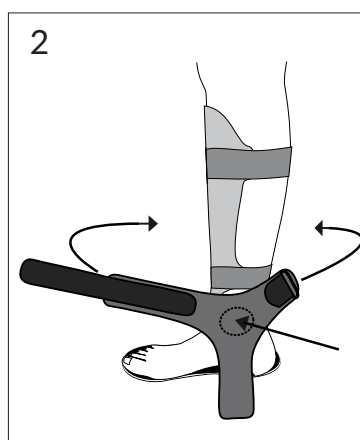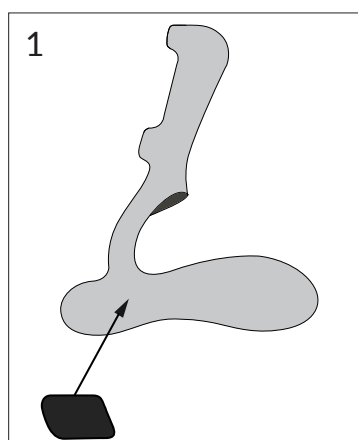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©Allard International_0525_rev.01