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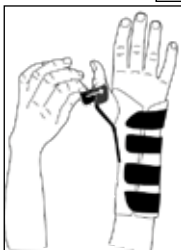
Carefully read this instruction sheet before use!

**Intended use/Indications**

This product is intended to restrict movements in CMC and MCPJ joints and to position the thumb in a functional position and to reduce flexion in the wrist. Indications such as Mild to Moderate CMC and MCPJ instability, Ligament Injuries, Tendinitis, Degenerative diseases, Trauma. Intended for one persons use.

**Contraindications**

No known

**Limitations**

This product should only be altered by a medical professional person. The prescriber will determine the duration and conditions for use.

**Warnings**

Make sure that the straps are not too tight. If any sign of skin irritation occurs, immediately remove the support. Check your skin on a daily basis.

**Material Specifications**

Made from cotton, nylon, polyester, lycra and polyurethane. Spoons made in Polypropylene and Manganese. This product is not flame retardant. Latex free.

**Washing Instruction**

Close all Velcro prior to laundering. Machine wash in 40°C (104°F), gentle cycle. Use a washing bag. Dry flat. Do not tumble dry or dry clean.

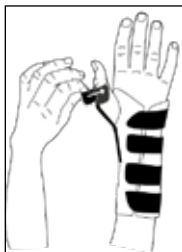
**Who to Contact**

Contact your fitter or store where purchased if you have questions regarding the use of the product.

---

## Instructions for use

1. Check that the product is complete and without damage.
2. Open the four wide Velcro straps.
3. Place the orthosis over the thumb and make sure that the spoon is correct positioned in the palm of the hand.
4. Close the four wide Velcro straps.
5. Adjust the thumb strap.
6. If necessary shape the spoon on the thumb side.
7. Adjust all straps for comfort.

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