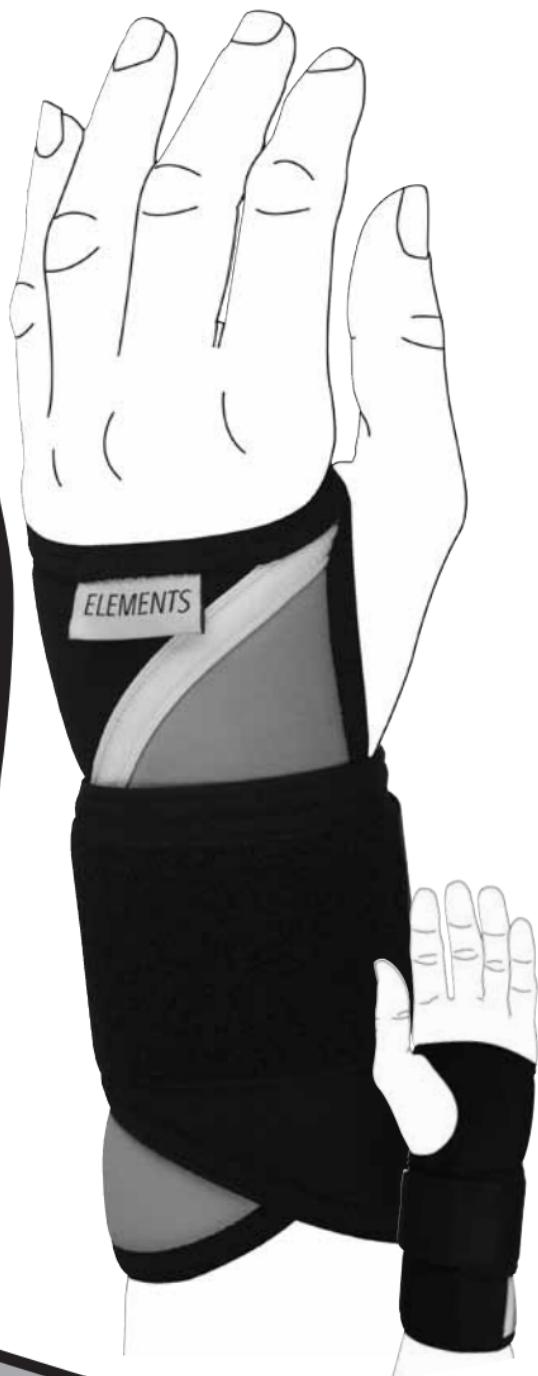
Cl_s

ELEMENTS

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Indications / Intended Use

This product is intended for treatment of mild and temporary conditions such as pain, inflammation and joint stress. The support limits the movement of the wrist, due to the aluminium stay on the palmar side. The support keeps the wrist warm. This support is only intended for one person use.

Contraindications

Severe edema, Severe Swelling.

Adjustments

This product should only be altered by a medical professional person.

Warnings

Check that you have the correct size. Some conditions can result in swelling that can mean variations in size. If this is the first time you are using a support it is important to start slowly. Use the support for a short period of time the first day, then increase the use gradually. The support should not be used during overnight. Do not use directly on open

wounds. Use only on dry skin. Do not use lotion on your skin. Make sure the straps are not too tight. If any sign of skin irritation occurs, immediately remove the support. Check your skin on a daily basis.

Material Specifications

This product is made from: Polyamid, polyurethane and spandex with an aluminium support. This product does not contain Latex, and is not flame retardant.

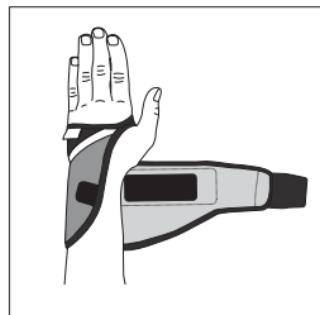
Washing Instruction

Close all Velcro prior to laundering. Machine wash in 40°C (104°F). Use a washing bag. Dry flat. Do not tumble dry or dry clean.

Who to Contact

If any problems occur, remove the support and contact your fitter or store where purchased.

Instructions for Use

1. Check that the support is complete and without damage.

2. Open the Velcro for the wrist.

3. Place the opening over the fingers with the aluminium support comfortably placed inside the hand.

4. Pull the strap around the wrist and close the Velcro.

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