28548, 28558
28549, 28559
THUMB
ORTHOSIS

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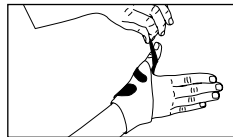
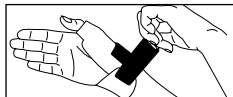
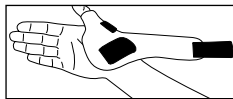
ENGLISH

28548, 28558, 28549, 28559 THUMB ORTHOSIS

Carefully read this instruction sheet before use!

Indications / Intended Use

This product is intended to protect the CMC and MCP-I joints, and is intended mainly for CMC-I arthrosis. It can also be used for rheumatic arthritis, after trauma, inflammation, joint stress and for rheumatic degenerations. The main function is to limit movement and give warmth.

Contraindications

No known.

Adjustments

This product should only be altered by a medical professional person.

Warnings

Make sure that the straps are not too tight. If any sign of skin irritation occurs, immediately remove the support.

Check you skin on a daily basis.

Material Specifications

This orthosis is made from Neoprene with a nylon surface. This product is not flame retardant

Washing Instruction

Close all Velcro prior to laundering. Machine wash in 40°C (104°F). Use a washing bag. Dry flat. Do not tumble dry or dry clean.

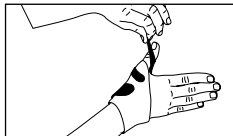
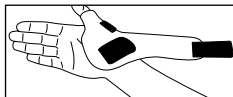
Who to Contact

If any problems occur, remove the support and contact your fitter or store where purchased.

Instructions for use

1. Check that the support is complete and without damage.
2. Attach the thumb strap and place the opening over the thumb with the orthosis towards the back of the hand.
3. Pull the strap around the wrist and secure.
4. Adjust the thumb strap so it pulls the thumb into a pain free position.
5. Adjust for comfort.

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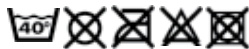
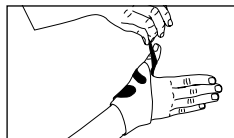
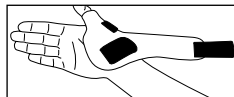
Support For Better Life • www.allardsupport.com

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Camp Scandinavia AB, Karbingatan 38