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Fast-LITE™

Instructions for use - EN

Description Fast-Lite orthoses (splints) are pre-formed to be used in the treatment of acute fractures and injuries requiring immobilisation. They are made from X Lite classic material.
Intended Use and Indications
1.The pre-shaped Fast-Lite orthosis is intended to immobilise body parts after an injury or pre- and post-surgery, for a limited time.
2. The Fast-Lite orthosis should only be applied by a medical professional who has been trained in casting or splinting.
3. The orthosis has a shelf-life of 5 years – please use by date on the packaging.
Instructions for Use
1.Apply the appropriate amount of stockinette and padding to the affected limb for patient comfort, to allow for swelling and to protect the patient’s skin and bony prominences.
2. Soften the Fast-Lite orthosis to create the best fit by applying heat at approximately 65°C for 30 seconds. (Ideally use a splint bath).
3. Start by heating the elbow or ankle section until it is completely soft and then heat the rest of the orthosis until it is mouldable but not out of shape.
4. Use tongs to remove the orthosis from the water.
5. Remove excess water by shaking the orthosis over the splint bath.
6. Start the application at the ankle/elbow section and once a good fit has been achieved in this area move to the other parts of the orthosis.
7. Apply to the affected limb and allow to set (for approximately 2 minutes) before fixing the orthosis in place with an elasticated bandage.
8. The orthosis may be trimmed to size to provide a better fit if required. (Please note that the orthosis can be trimmed to size before, during or after fitting)
Precautions and Warnings
1. Take care when applying heat to the orthosis and when removing from hot water.
2. Fast-Lite orthoses are fully set after approximately 30 minutes.
3. Inform the patient to avoid warm temperatures and strong sunshine.
4. Fast-Lite material is sensitive to chlorinated water.
5. A Fast-Lite orthosis can be used multiple times on the same patient.
6. Lower leg Fast-Lite orthoses are not designed for full weight bearing (eg walking).
7. Residues of polymer can appear in the water bath and around the workstation. Remove with warm water and mild cleaner.
Storage
1. Store in a dry, dark place at temperatures between 5°C (41° F) and 40°C (104°F).
Disposal
1. Splints may be incinerated or sent to landfill in accordance with local regulations.
Notice
1. Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.
Material specification
1. Fast-Lite contains cotton and polycaprolactone. Latex free.

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