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ENG

The Boston Soft AFO is an inexpensive alternative to custom AFO's. It can be used on either the left or right foot. It is lightweight, comfortable and fully adjustable.

The Soft AFO is used for non-ambulatory patients to enhance foot positioning. The Soft AFO is available in six Paediatric sizes and six adult sizes. An optional anti-rotational stabilizer is also available if required.

Indications

The Boston Soft AFO is useful for a number of conditions including:

Myelomeningocele

Muscular Dystrophy

Cerebral Palsy

Trauma

Heel Ulcerations

Post-Surgery

Measurement for AFO

The Soft AFO is selected based on the patient's shoe size.

Fitting the Soft AFO

It is advisable to wear a high cotton sock under the Soft AFO. If the patient has an AFO for each foot mark them as they may require slightly different adjustments for each foot.


When putting on the AFO bend the patient's knee to 90° and push their heel back into the AFO. It can help to gently lift the toes upward to check that the heel is right down in the heel section of the AFO.

Fasten the ankle strap first, then the calf strap and finally the foot strap. Make sure all the straps are fastened securely. If the AFO needs to be trimmed mark the trim line on the AFO with a pen/pencil, remove the AFO and using a pair of scissors trim off the excess material.

After the patient has been wearing the AFO for a short period of time remove the AFO and check the skin for any pressure marks.

Care of AFO

The Soft AFO can be cleaned with a soap and water and dried with a towel or left to dry naturally.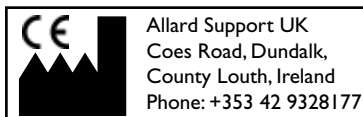

Distributed in Scandinavia by:
Camp Scandinavia AB
Karbingatan 38, 254 67 Helsingborg,
Sweden
Phone: 46 42 25 27 01